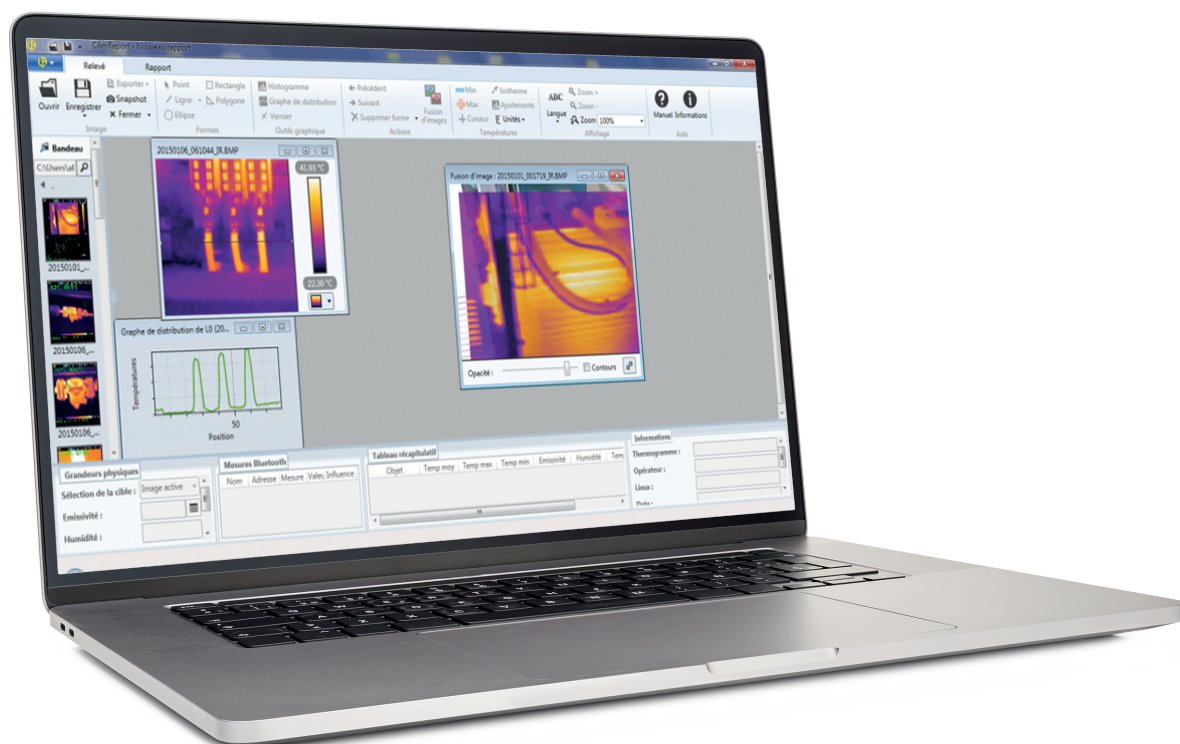
CAmReport AI

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The figure shows two screenshots of the CamReport software interface, specifically the 'Nouveau rapport' (New report) screen. The interface is divided into several sections: 'Title', 'Business', and 'Writing'. The 'Business' section includes fields for 'Company', 'Telephone', and 'Address'. The 'Writing' section includes fields for 'Author', 'Telephone', and 'Mail'. A 'Comments' field is also present at the bottom. The left screenshot shows the initial state with empty fields. The right screenshot shows the form filled with data: 'Title' is 'RCD', 'Company' is 'Chauvin Arnoux', 'Telephone' is '+33 1 44 85 44 85', 'Address' is '12-16 rue Sarah Bernhardt 92600 Asnières-sur-Seine', 'Author' is 'John Smith', and 'Mail' is 'info@chauvin-arnoux.com'. A red arrow points from the left screenshot to the right screenshot, indicating a transition or completion of the form.

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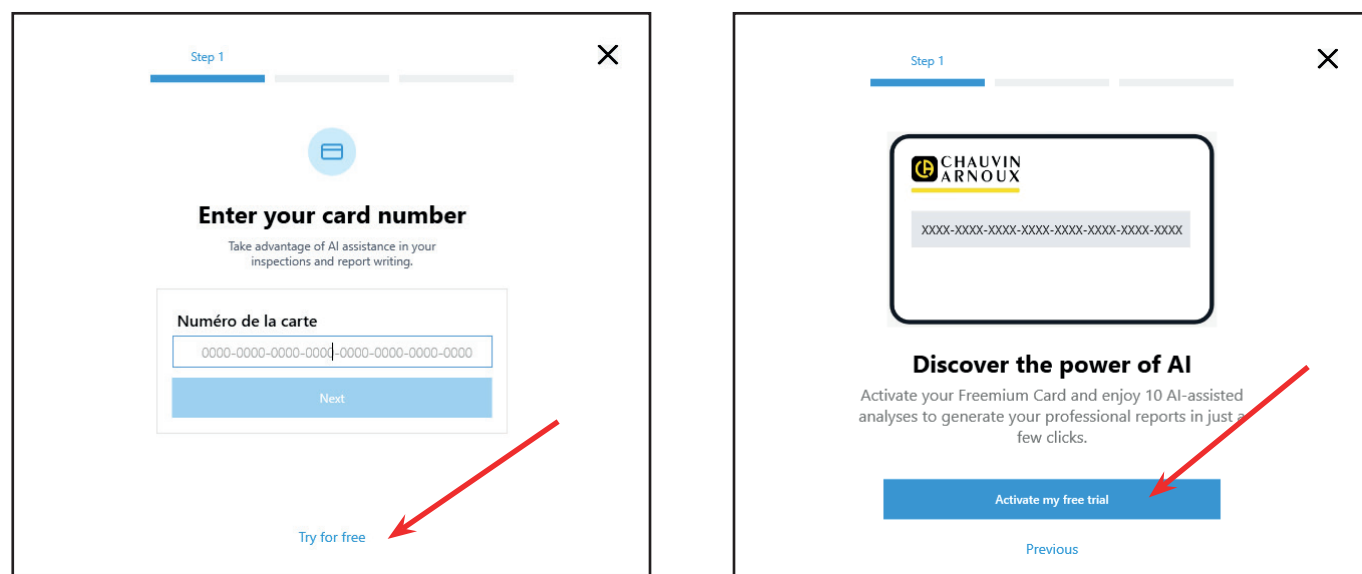
Figura 19

Step 1

Please identify your camera using a thermal image to start your free trial.

Select a thermal image taken with your Chauvin Arnoux camera.
If your camera is eligible, you will receive 10 free AI analyses.

Browse

Next

Step 2

Last Name* First Name*

Company* Activity

Email* Phone*

Address

Postal Code* City* Country*

☐ I have read and accept the [The terms and conditions of use](#)*

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Step 3

Verify your email

We have sent a 6-digit verification code to your email address. Please enter it below.

Verify

Didn't receive a code? [Resend code](#)

← Return

CHAUVIN ARNOUX

1000 AI assisted analyses

Card Starter

Valid until: 01/12/2026

Your card has been successfully activated!

You can now access AI-assisted analyses on your thermal images.

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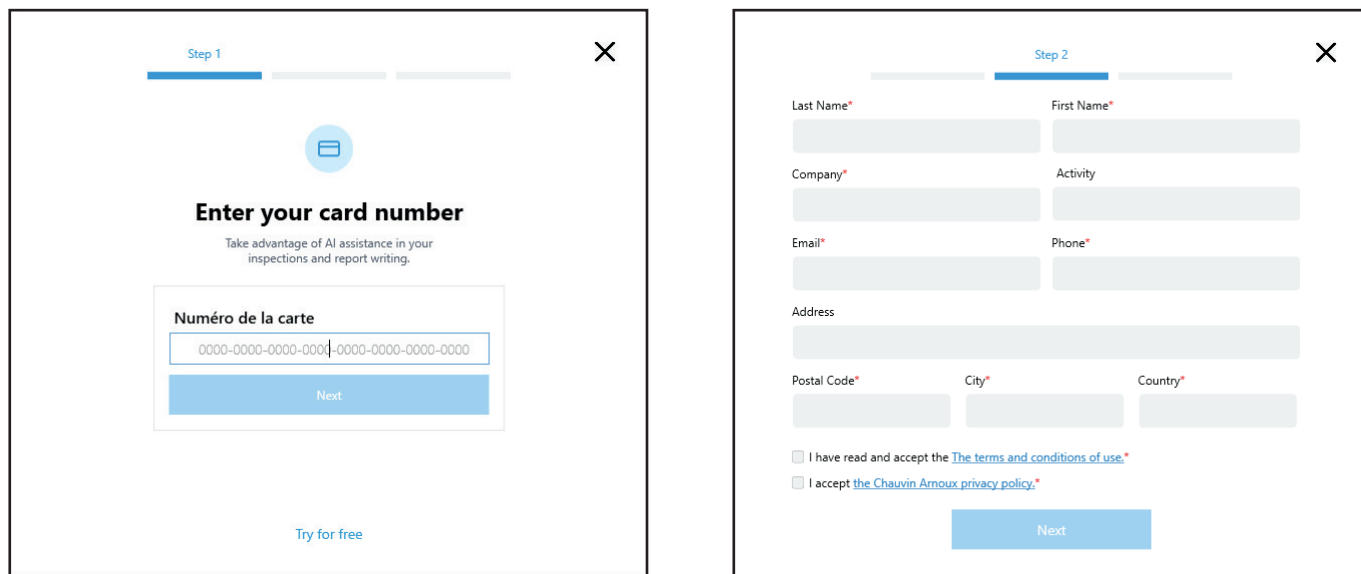
Step 1

Enter your card number

Take advantage of AI assistance in your inspections and report writing.

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0000-0000-0000-0000-0000-0000-0000-0000

Next

Try for free

Step 2

Last Name* First Name*

Company* Activity

Email* Phone*

Address

Postal Code* City* Country*

☐ I have read and accept the [The terms and conditions of use.](#)*

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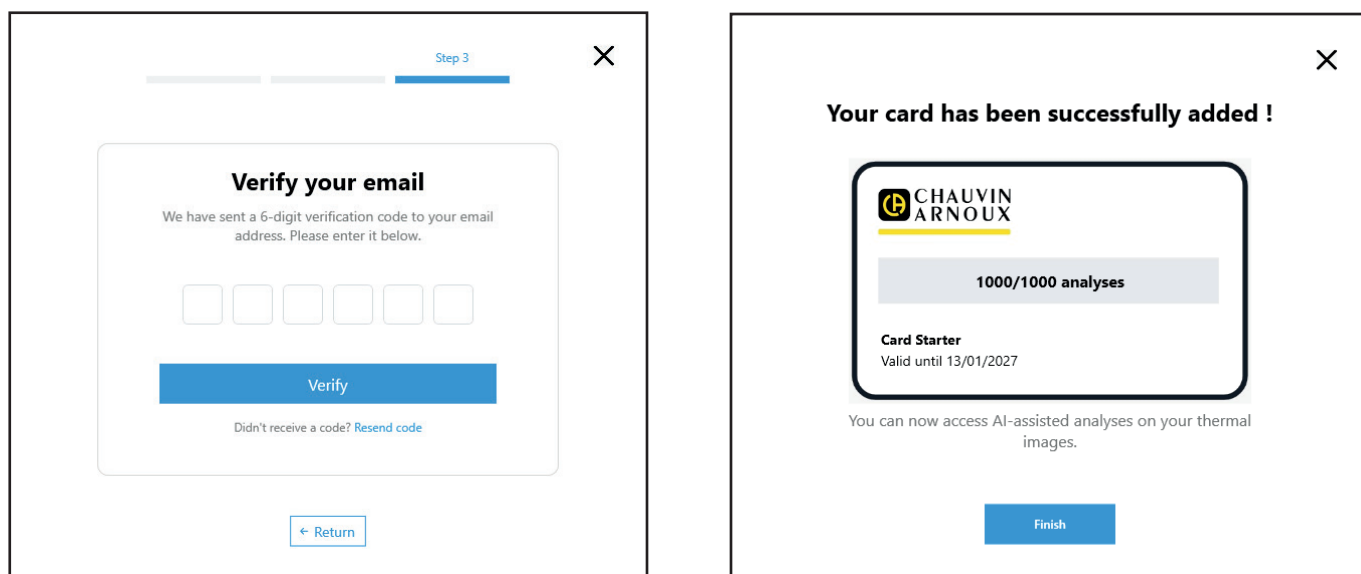
Step 3

Verify your email

We have sent a 6-digit verification code to your email address. Please enter it below.

Verify

Didn't receive a code? [Resend code](#)

← Return

Your card has been successfully added !

 CHAUVIN ARNOUX

1000/1000 analyses

Card Starter

Valid until 13/01/2027

You can now access AI-assisted analyses on your thermal images.

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
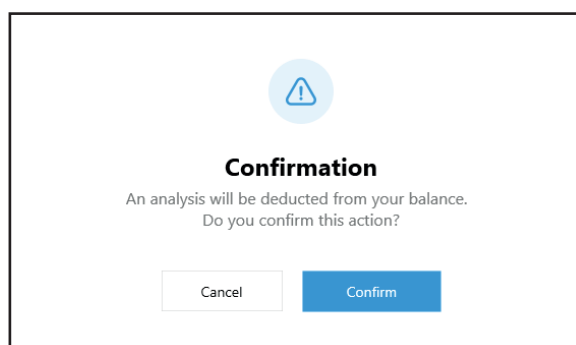
Figura 25

AI-assisted report creation

Below is the analysis result. You can customize it before inserting it into your inspection report.

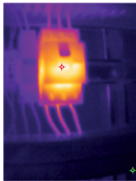Analyzed images:

20180215_100922_IR.png

20180215_100922.png

Comments :

The thermal image shows a significant temperature difference on a component, identified as a circuit breaker in the visible-light image. The upper part of the breaker exhibits a maximum temperature of 57.2°C, while the ambient temperature is 25°C, resulting in a ΔT of 32.2°C. The rest of the breaker and adjacent components show much lower temperatures, indicating a localised heat source, likely due to an internal fault or high resistance at a connection.

Recommendations :

Immediate action is required to disconnect the circuit breaker and inspect it for internal faults or loose connections. If necessary, replace the breaker to prevent potential failure or fire hazard. Ensure all connections are properly tightened and recheck with thermographic inspection after repairs.

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Analysis
Report

Open Save Export Print Template

ABC Language History My card Manual information Help

Presentation
Resources
Analysis
Preview

← Previous inspection
20180215_103012_IR.png
Next inspection →

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Inspection name:

Operator:

Location:

Date taken: 15/02/2018 10:30

Equipment:

Comments :

The provided thermal image displays a section of an electrical panel containing circuit breakers. The thermal pattern indicates elevated temperatures on the upper section of one breaker, with a peak temperature of 50.2°C, compared to surrounding areas. The visible temperature gradient suggests potential excess heat generation at specific points, indicating potential electrical or mechanical issues.

Recommendations :

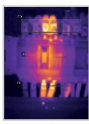
Immediate inspection of the highlighted breaker should be conducted, focusing on connections and load conditions. Implement periodic thermal imaging assessments with a monthly frequency to monitor for further temperature elevation. Ensure personal protective equipment is utilized during interventions due to potential electrical hazards.

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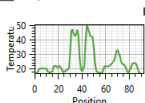
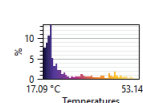
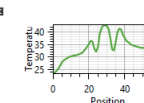
Analysis history

Analyses from most recent to oldest

Date and time ↓	Image	Preview	Balance debited
13/01/2026 09:17:31	20180215_100922		1
13/01/2026 09:15:20	20150106_013208		1
13/01/2026 08:36:14	20150101_000110_IR - Copie		1
13/01/2026 08:32:06	20180215_100922		1
13/01/2026 08:29:26	20151215_090229a		1

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Analysis of 20150106_013208 ID Adjusted ×

13/01/2026 09:15:20 • Balance debited : 1 • The remaining balance at the time of this analysis : 996

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Mineure

[Show original](#)

Important: AI-generated results may contain errors and require verification. Any recommended actions must be performed by a qualified professional.

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